



MesaLabs

INSTRUCTIONS FOR USE

ProChem EXT

1. Place ProChem EXT in the most difficult to sterilize area of every package/tray to be sterilized. For larger packs, use multiple strips. Do not use if indicator strip is not green prior to exposure to steam.
2. Run steam sterilization cycle as usual.
3. After cycle is complete, visually verify indicator strip has changed from green to black, becoming as dark or darker in intensity than reference arrow. This change is indicative that sterilization conditions were achieved.
4. If strip is green or not as dark or darker than the reference arrow, consider the cycle failed and the items non-sterile. Follow in-house protocol for failed cycle.

For use at 121°C or 132-135°C