


Exposure® Biological Indicator

ExpoSure® Biological Indicator

ExpoSure® is a biological indicator intended for use in the STERRAD® Sterilizer.

ExpoSure has received its 510(k) making it suitable for healthcare and non-healthcare settings and usage. Its 24-hour incubation time and flexibility make it a powerful tool to help your facility maintain compliance and patient safety.

ExpoSure is a direct replacement product and can be used in conjunction with existing incubators out in the field (55-60 °C). The on-board chemical indicator confirms that it has been exposed to hydrogen peroxide by changing from red to blue upon exposure.

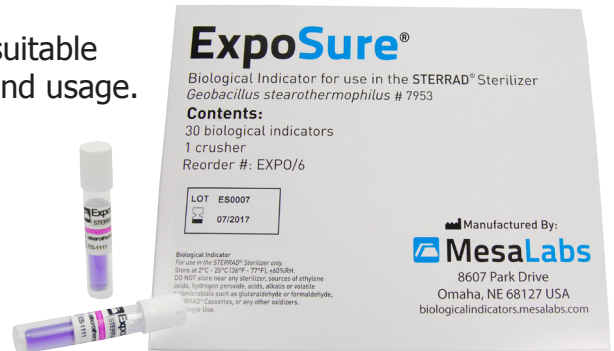

Packaging:

ExpoSure is packaged with 30 indicators per box along with 1 activation crusher.

Reorder #:

EXPO/6

Expiry:

12 months

Approved Cycles:

- STERRAD® 50
- STERRAD® 100S (Short & Long)
- STERRAD® 200 (Short & Long)
- STERRAD® NX® (Advanced & Standard)
- STERRAD® 100NX® (STANDARD, FLEX, DUO, and EXPRESS)

Incubation:

24 hours at 55-60 °C: Mesa offers its [1410 Dry Bath Incubator](#) for use with ExpoSure



Placement:

Place the ExpoSure® BI in a Tyvek® pouch and place the pouch in the most challenging area for the sterilant to reach. This is typically on the shelf close to the rear of the sterilizer.